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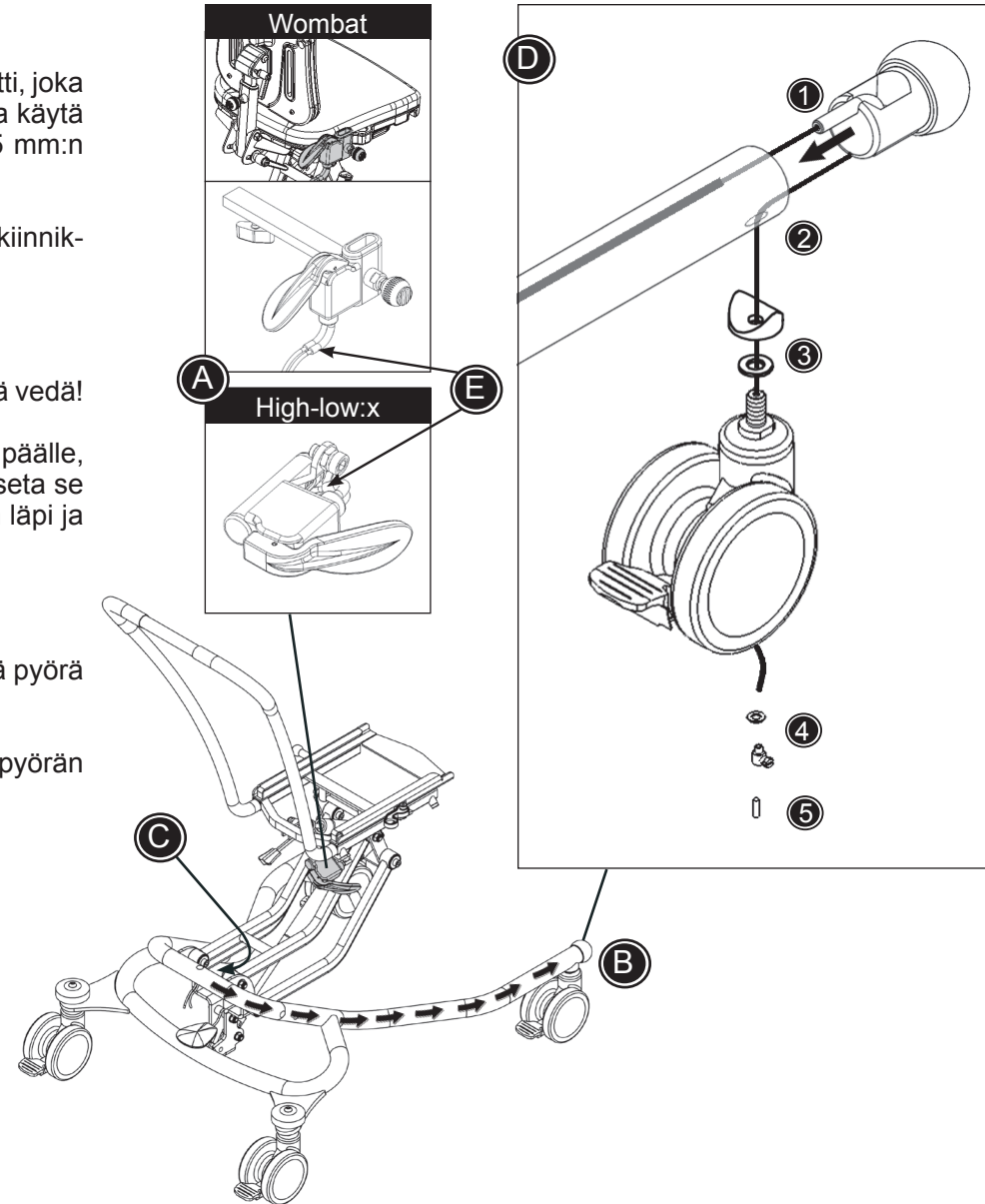
# Mounting instruction

R82<sup>®</sup>

## Mounting a central brake on High-low:x and Wombat

1. Mounting a central brake handle (A) on High-low:x: Remove original bolt which locks the push brace into position and mount the enclosed counter nut on the bolt and use it to mount the handle in the same position. Tighten both the screw and the counter nut with a 5 mm Allen key and 13 mm spanner.

Mounting central brake handle (A) on Wombat: Mount the handle on the mounting-fitting and slide it under the seat and fasten with finger screw from beneath.

2. Dismount a front wheel with a 13 mm spanner and remove the plastic plug (B)
3. Pass a brake cable through the frame (C) and out by one of the front wheels. Push the cable - *do not pull!*

4. Mount cable and wheel like this (D): Pull the small tube (1) on to the brake cable, pull the cable down through the plastic plug and place the small tube in the slit. Pull the cable back and place in the slit under the plastic plug. Pull the cable end through the hole in the frame (2) and push the plastic plug back into the frame.

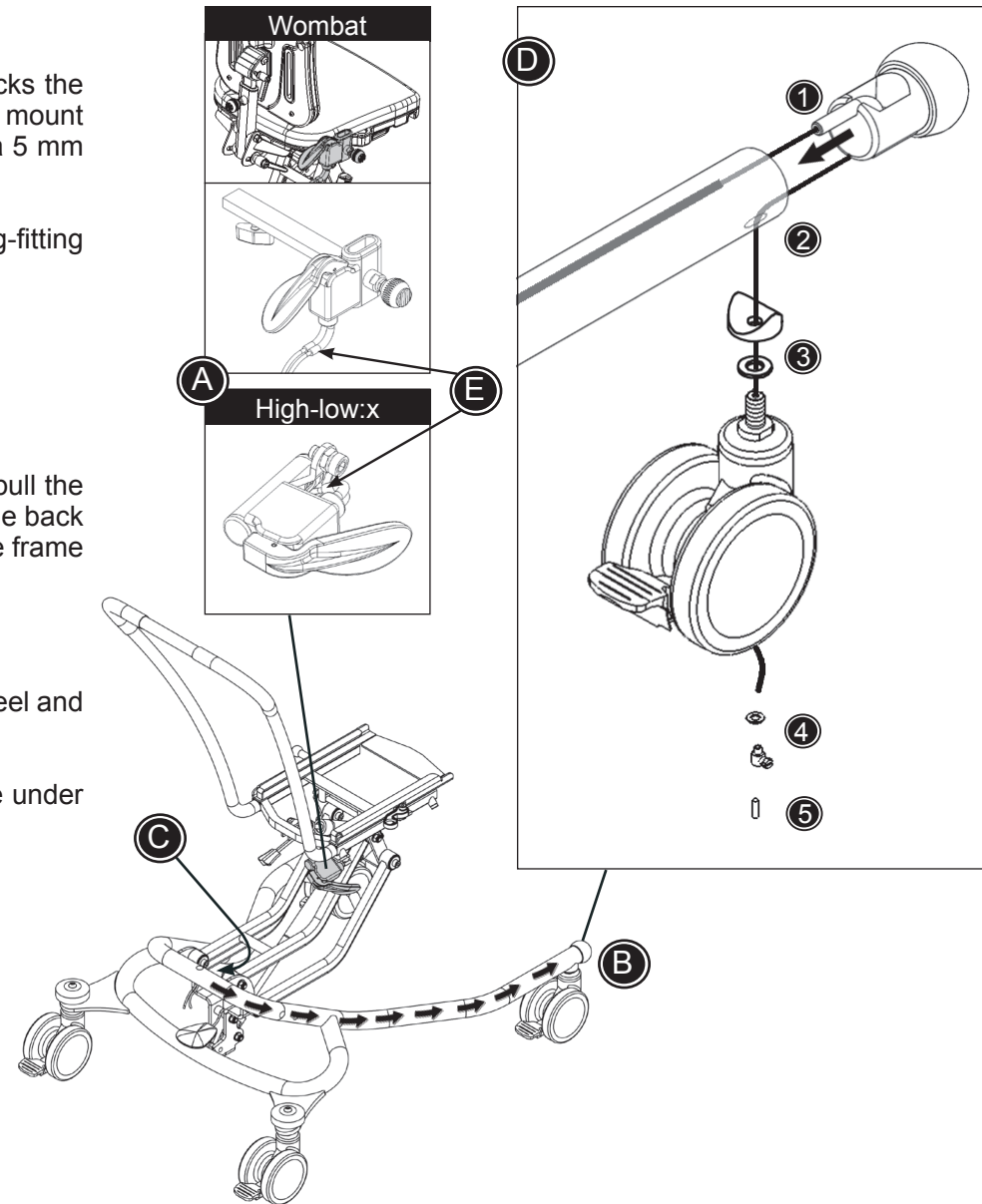
Push the central brake handle to make sure that the brake cable moves.

Mount original fittings (3) on the new wheel and pull the brake cable through the wheel and fasten the wheel with a 13 mm spanner.

Pull the brake cable so it tightens. Mount washer and cable clamp (4) on the cable under the wheel to maintain a tight cable.

*Complete step 2-4 at both front wheels.*

5. Test that the brake is locking as desired. Adjust the strength of the brake on the cable adjuster (E): Loosen the cable adjuster to tighten the brake, and tighten the cable adjuster to loosen the brake. Adjust both cables and tighten the counter nuts on the cable tighteners.
6. Cut the brake cable about 5 mm from the cable clamp (under both wheels) (4) and fix the nipple (5) on the cable end with a pliers.

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