

etac[®]
Creating Possibilities



R82[®]

Chilla

Designed to enhance comfort, stability, and safety for your child

Designed for adventure

From city streets to nature trails and every adventure in between, the R82 Chilla is designed to grow with your child - offering growth, comfort, support, and freedom to explore the world together.

With both front- and rear-facing options, the Chilla lets your child take in their surroundings or stay connected with you. It's EASY-FOLD design allows for effortless folding, even with the seat and accessories attached. Need to save space? The frame and seat can also fold separately to fit into smaller spaces, providing you maximum flexibility on the go.



Practicality, built-in growth and tailored support

Chilla grows with your child, accommodating their changing needs over the years. With adjustable seat length and back height, coupled with a variety of accessories, Chilla is the perfect partner.

Its innovative design incorporates a range of accessories, enhancing both functionality and convenience. With options for the frame to carry essential items, and accessories for the seat to provide optimal support, comfort and protection from the weather, Chilla ensures practicality, comfort and support on every outing.



Adventure ready and transit approved

Chilla is crash tested and complies with ISO 7176-19, for safe transportation in motor vehicles. All you need to do is mount the transport fittings for securing Chilla in a motor vehicle, and you are ready for the next adventure.

Chilla has soft, thick cushions for added support and comfort. Combined with a vest and belt and the adjustable suspension, Chilla provides a secure, supportive and comfortable ride.

Toolless adjustments for everyday use

Chilla is designed with the caregiver and child in mind. With a toolless back recline option you can easily change between upright and the horizontal positions. Posterior tilt and back recline reduce the effect of gravity and can therefore be helpful to decrease fatigue and conserve energy, so that the child has more energy for participating in fun activities throughout the day.

The Chilla seat can be both front and rear facing, so the child can face its surroundings or have the comfort of being able to see its parents. Seat reversal requires only a single operation of the safety lock and tilt trigger at the back of the seat. No tools required.



Ready for nature - Advanced suspension system

Optional adjustable suspension improves stability when moving over terrain and helps to absorb vibrations and sudden shocks during movement, providing a more comfortable and steadier ride.

Chilla's advanced suspension features true damping, meaning it quickly returns to its natural equilibrium position without the jolts or bouncing of traditional systems. This smoother, controlled response helps absorb bumps from uneven terrain, creating a more comfortable and stable ride for your child.

Offering three preset settings so you can customize the ride to your child's environment, needs and daily activities. Whether navigating different terrains or accommodating additional weight - like an oxygen bottle - preloading the spring helps balance the stroller for a smoother, more stable ride and effortless maneuverability.



Product highlights

Offering a range of features and options to make it possible for every child to receive tailored support while exploring their world.

Convenient Storage

The Under Storage Basket comes at no additional cost and is the perfect size for storing your daily essentials.

Adjustable suspension

The Chilla comes with the option of adjustable suspension on the frame. The suspension on Chilla has, what is called, true damping, which means that once the suspension is loaded, it will return to its original position smoothly and without bouncing.

Early intervention

The infant cushion for size 1 provides supported seating for infants, thereby giving the opportunity for early intervention and facilitation of the child's participation in activities and social interaction with parents, carers, and the environment.

Outdoor adaptable

Chilla is equipped to handle a wide range of weather conditions. An optional rain cover is available to shield against wind, rain and snow. Additionally, you can choose between a standard canopy and a premium canopy for added versatility and protection.

Going active or planning a nap?

The Chilla stroller offers tilt-in-space, which is possible to operate with just one hand, even when the child is seated. Chilla is designed with a high focus on a centred rotation axis for the tilt. This implies that the tilt function will be as smooth as possible and requires minimum force to operate.

Growth and support

Chilla adapts effortlessly, thanks to its adjustable seat depth and back height. And if you need to adjust the seat or back width, you can add hip and side supports for a snug and secure fit. This flexibility guarantees that your child enjoys a stable seating position, fostering interaction and participation.

Cushions and covers

The cushion for Chilla comes in four different colors: grey, red, blue and pink. Pink is only available for Chilla size 1. The grey, blue and red cushion options are also available in mesh to provide a softer base for the child.



[View Full Option Offering](#)

[Go to video playlist with the Chilla](#)





More
Information

Measurements

		1 Normal	1 Suspension	2 Normal	2 Suspension
	Effective seat width	13"	13"	15"	15"
	Effective seat depth	7" - 13.5"	7" - 13.5"	9.5" - 16.5"	9.5" - 16.5"
	Backrest height with head support plate	20" - 26.25"	20" - 26.25"	24.5" - 32"	24.5" - 32"
	Backrest height without head support plate	14.5" - 20.75"	14.5" - 20.75"	16.5" - 24"	16.5" - 24"
	Seat surface height at front edge	17.75"	17.75"	17.75"	17.75"
	Max. load	94lbs	94lbs	132lbs	132lbs
	Max user mass in transportation	77lbs	77lbs	110lbs	110lbs
	Max user mass in transportation	77lbs	77lbs	110lbs	110lbs
	Overall height to pushbrace	47.25"	47.25"	47.25"	47.25"
	Length	28.75"	30"	28.75"	30"
	Overall width	23.5"	24.5"	26"	26.75"
	Folded length	40.75"	40.75"	40.75"	40.75"
	Folded height	15"	15"	15"	15"
	Mass device	20lbs	21lbs	20lbs	22lbs
	Mass device	11.7lbs	11.7lbs	13.8lbs	13.8lbs

*Further measurements can be found online

A splash of color



Grey cushions

For size 1 and 2 - also comes in a mesh option.



Red cushions

For size 1 and 2 - also comes in a mesh option.



Blue cushions

For size 1 and 2 - also comes in a mesh option.



Pink cushions

Only for size 1.



Chilla_Prod... 01/2021 | 2021 | Reserving our rights for printer's error or discontinued products. Please find an updated edition on our website etac.com
© R82 A/S All rights reserved.

Etac is a world-leading developer and provider of ergonomic assistive devices and patient handling equipment. Our heart lies in the solutions that optimize quality of life for the individual, their family and caregivers.

For the latest news and continuously updated product information – visit: www.etac.us.com

Etac
+1 844-US-MOBILITY (844-876-6245)
sales.us@etac.com
www.etac.us.com
DCN1331.0


Creating Possibilities