

**etac**<sup>®</sup>  
Creating Possibilities



**etac**<sup>®</sup>

# Etac Swift Commode

A three-in-one solution



# Swift Commode

Swift Commode is a versatile product that can be used as a commode, shower chair or toilet seat raiser. It is excellent for people who need access to a toilet near the bed, since it provides functionality combined with a discreet design for the bedroom. The commode is just as suitable in the bathroom, as it is can sustain wet environments and be used in the shower as well as over the toilet.

## Three-in-one functionality

In the bedroom, Swift Commode functions as a nearby toilet. Designed like an armchair, the soft pads for seat and back support are removable and conceal a pan with a self-locking lid. The commode is equally suited for the bathroom, where it can be used as a shower chair with or without the pads. When placed over the toilet, it functions as a free-standing toilet seat raiser. The chair can easily be adjusted to the preferred height, and the arm supports offer a stable support when standing up or sitting down.

## Designed for flexibility

Swift Commode is light in weight and easy to move between the bedroom and bathroom. At the same time, it can accommodate larger users with a maximum user weight of 160 kg. It is height-adjustable between 42-57 cm, and can be set with a slight anterior tilt to further facilitate standing up.

## Easy assembly and cleaning


Swift is easy to assemble without the need for any tools. Arm and back supports can just as easily be removed if needed, which also simplifies cleaning. The chair has smooth surfaces and is non-corroding. The seat pad, back support pad and pan are included as standard.

Swift Commode comes with a 5-year warranty and a 10-year service life.



<b>Swift Commode</b>	<b>Item no.</b>	<b>Weight</b>	
Swift Commode Swift Commode basic without pads	81702030 81702060	5.4 kg (12 lbs)	
<b>Dimensions</b>	<b>Maximum user weight</b>	160 kg (353 lbs)	
Total width: 56 cm (22") Seat width: 54 cm (21¼") Seat height 42-57 cm (16½-22½") Width between the arm supports: 45 cm (17¾")	<b>Maintenance</b>		
<b>Materials</b>	Clean / disinfect the product using a standard non-abrasive cleaning agent with a pH between 5 and 9, or with a 70% disinfectant solution. The product can be safely cleaned in a cabinet washer-disinfector at 85°C for 3 minutes.		
Seat, back and arm supports: polypropylene Legs: aluminum and polyamide Ferrules: TPE Back and seat pads: Polyethylene with nylon fabric			

## Features

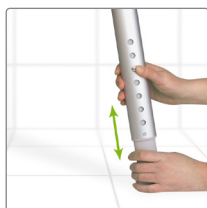

### Smart height adjustment

The telescopic legs can swiftly be adjusted to the preferred height thanks to a self-locking button that clicks into place. The oval shape prevents the legs from turning when making the adjustment. There are no loose parts to misplace.



### Dependably stable

The chair has a large footprint which makes it extremely stable, and a generous maximum user weight of 160 kg. The legs' responsive, non-slip ferrules adapt to the floor and provide an excellent grip on wet or uneven surfaces.



### Easy handling

Swift Commode comes flat-packed in a smart carry case. Since the chair is easily assembled and disassembled, the case can be saved and used for further transports if the chair is used in several venues or settings.



### Pan with integrated, self-locking lid

The pan with a tight-locking lid, fits smoothly and becomes one with the seat. The pan has a handle in the back as well as an upright handle that automatically locks the lid when carried.



Swift Commode without pan



Swift Commode with pan



Swift Commode with pads

## Accessories



### Toilet paper holder

80301629

A flexible paper holder that is attached to the arm support. Can be placed on both left and right arm support and with the opening facing forward or backward.

Material: Polyamide, stainless steel



### Seat pad

84005064

The non-slip seat pad provides an instant warm feeling and makes sitting more comfortable. Easy to wipe down.

Material: Polyethylene



### Soap dish / Lid holder

80209266

Attaches to the side of the seat. Perfect place to store shower gel, shampoo, pan lid. Also equipped with a built-in shower holder.

Material: Polypropylene



### Back support pad

84005065

Fitted on the back support for extra comfort and provides a warm feeling. It is non-slip, simple to click into place and easy to clean. Included as standard.

Material: Polyethylene



### Splash guard

80209430

The splash guard provides a soft protection in the front to avoid splashing. Attaches to the front of the seat and can easily be removed.

Material: Polyurethane



Swift Commode - leaflet\_01\_INT

Etac is a world-leading developer of innovative assistive devices and patient handling equipment. Since 1973, we have been committed to improving quality of life for the individual, family members and caregivers.

For the intended purpose of the products and latest information, please visit [www.etac.com](http://www.etac.com)

Etac

+46 371 587 330

info@etac.se

[www.etac.com](http://www.etac.com)

**etac**<sup>®</sup>  
Creating Possibilities