

**etac**<sup>®</sup>  
Creating Possibilities



**etac**<sup>®</sup>

# Etac Swift Freestanding

Toilet seat raiser

# Swift Freestanding toilet seat raiser

Swift Freestanding is the perfect choice when you want a raised toilet seat easily and quickly. By simply setting it to the preferred height and placing it over the toilet, you have a seat height adapted to your needs, and sturdy arm supports that facilitate standing up and sitting down. When complemented with a pan, it can just as easily be used in the bedroom as a stand-alone toilet chair.

## Designed for flexibility

In addition, Swift Freestanding can serve as a shower stool. Its light weight makes it easy to move and it is designed to sustain wet environments. The stool is extremely stable thanks to its large footprint and non-slip ferrules that provide an excellent grip on wet or uneven surfaces. With a maximum user weight of 160 kg it accommodates larger users well.

Swift's seat is height-adjustable from 42-57 cm, and can be set with a slight anterior tilt which further facilitates standing up and relieves pressure on the hip joints.

## Functions that can evolve over time

Apart from being complemented with a pan, Swift Freestanding can also be fitted with a back support and paddings to provide extra comfort and support. This way it can gradually change over time, along with the needs of the user.

## Easy assembly and cleaning

Swift Freestanding is easy to assemble without the need for any tools. The arm supports and legs can just as easily be removed, which is great for car transportation and simplifies cleaning. The chair has an integrated splash guard, smooth surfaces and is non-corroding.




Swift Freestanding comes with a 5-year warranty and a 10-year service life.

<p><b>Swift Freestanding</b></p> <p>Swift Freestanding toilet seat raiser</p>	<p><b>Item no.</b></p> <p>81702020</p>	<p><b>Weight</b></p> <p>3.9 kg (8.6 lbs)</p>	
<p><b>Dimensions</b></p> <p>Total width: 56 cm (22")</p> <p>Seat width: 54 cm (21¼")</p> <p>Seat height 42-57 cm (16½-22½")</p> <p>Width between the arm supports: 45 cm (17¾")</p>	<p><b>Maximum user weight</b></p> <p>160 kg (353 lbs)</p>	<p><b>Maintenance</b></p>	
<p><b>Materials</b></p> <p>Seat and arm supports: polypropylene</p> <p>Legs: aluminum and polyamide</p> <p>Ferrules: TPE</p>	<p>Clean / disinfect the product using a standard non-abrasive cleaning agent with a pH between 5 and 9, or with a 70% disinfectant solution. The product can be safely cleaned in a cabinet washer-disinfector at 85°C for 3 minutes.</p>		



## Features

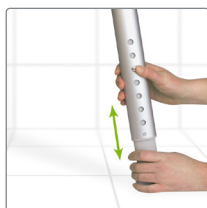

### Smart height adjustment

The telescopic legs can swiftly be adjusted to the preferred height thanks to a self-locking button that clicks into place. The oval shape prevents the legs from turning when making the adjustment. There are no loose parts to misplace.



### Dependably stable

The chair has a large footprint which makes it extremely stable, and a generous maximum user weight of 160 kg. The legs' responsive, non-slip ferrules adapt to the floor and provide an excellent grip on wet or uneven surfaces.



### Easy handling

Swift Freestanding comes flat-packed in a smart carry case. Since the chair is easily assembled and disassembled, the case can be saved and used for further transports if the chair is used in several venues or settings.



### Pan with integrated, self-locking lid

A pan is available as an accessory. It has a tight-locking lid, fits smoothly and becomes one with the seat. The pan has a handle in the back as well as an upright handle that automatically locks the lid when carried.



Swift Freestanding toilet seat raiser



Swift Freestanding with added back support



Swift Freestanding with added back support and pan

## Accessories



### Back support

84006181  
Back support that can be added for extra support and comfort. Easy to click into place without the need for any tools. Just as easy to detach.  
Material: Polypropylene



### Toilet paper holder

80301629  
A flexible paper holder that is attached to the armrest. Can be placed on both left and right armrest and with the opening facing forward or backward.  
Material: Polyamide, stainless steel



### Seat pad

84005064  
The non-slip seat pad provides an instant warm feeling and makes sitting more comfortable. Easy to wipe down.  
Material: Polyethylene



### Soap dish / Lid holder

80209266  
Attaches to the side of the seat. Perfect place to store shower gel, shampoo, pan lid. Also equipped with a built-in shower holder.  
Material: Polypropylene



### Back support pad

84005065  
Fitted on the back support for extra comfort and provides a warm feeling. It is non-slip, simple to click into place and easy to clean.  
Material: Polyethylene



### Splash guard

80209430  
The splash guard provides a soft protection in the front to avoid splashing. Attaches to the front of the seat and can easily be removed.  
Material: Polyurethane



### Pan

Without lid: 84005066  
With lid: 84005068  
A pan with a smart, integrated lid that locks automatically when lifted. Fits smoothly and becomes one with the seat.  
Material: Polypropylene





Etac is a world-leading developer of innovative assistive devices and patient handling equipment. Since 1973, we have been committed to improving quality of life for the individual, family members and caregivers.

For the intended purpose of the products and latest information, please visit [www.etac.com](http://www.etac.com)

Etac

+46 371 587 330

info@etac.se

www.etac.com

**etac**<sup>®</sup>  
Creating Possibilities