

Crocodile									
User information appropriate for use by people with visual, reading or cognitive disability	The latest version of all instructions are always available on the website https://www.etic.com and can be printed in larger sizes								
Information available in pictogram	The instructions are made as pictograms								
Intended use and intended environment	<p>Intended purpose The product is an assistive device for walking, intended for children and adults with disabilities.</p> <p>Indications The device can be used by children and adults with disabilities, who experience limitations in their ability to walk independently. The target group for the device is based on functional ability and anthropometry. It is not based on a specific health condition or age. The user is the person walking in the device. The caregiver is the person operating the device. Never leave the user unattended in this device. Ensure permanent supervision by a caregiver. The use of the device will be discontinued more times during the day where the user will be sitting, standing or lying down in another assistive device or a conventional product.</p> <p>Contra indications There are no known contraindications.</p> <p>Precautions Special attention should be given to the considerations about the configuration of the device and the use of accessories for users with limitation in their:</p> <ul style="list-style-type: none"> • Sensory functions and the sensation of pain, consider extra checks of the pressure caused by the support provided by the product and its accessories. • ability to maintain a standing position, consider using accessories such as back and side support with a strap or a sling seat. • ability to maintain the position of their head, consider using an assistive device for walking that provide support to the positioning of the head. <p>Application areas The product is for indoor and outdoor use</p>								
Maintenance instructions	<p>Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.</p> <table border="1" style="width: 100%; border-collapse: collapse;"> <thead> <tr> <th style="text-align: left;">Frequency</th> <th style="text-align: left;">Description</th> </tr> </thead> <tbody> <tr> <td style="vertical-align: top;">1st Day</td> <td> <ul style="list-style-type: none"> - Read the instructions for use (IFU) carefully before use and store them in a safe place for future reference - Store the enclosed tools on/at the product </td> </tr> <tr> <td style="vertical-align: top;">Daily</td> <td> <ul style="list-style-type: none"> - Use a dry cloth to clean the product - Visual inspection. Repair or replace any dam-aged or worn-out parts - Ensure that all fasteners are in place and fit-ted correctly - Check all fittings and buckles for signs of worn out parts - Be aware of any MAX mark indications before adjustment </td> </tr> <tr> <td style="vertical-align: top;">Weekly</td> <td> <ul style="list-style-type: none"> - Check that all castors are moving freely and that all wheel locks/brakes are easily activated </td> </tr> </tbody> </table>	Frequency	Description	1 st Day	<ul style="list-style-type: none"> - Read the instructions for use (IFU) carefully before use and store them in a safe place for future reference - Store the enclosed tools on/at the product 	Daily	<ul style="list-style-type: none"> - Use a dry cloth to clean the product - Visual inspection. Repair or replace any dam-aged or worn-out parts - Ensure that all fasteners are in place and fit-ted correctly - Check all fittings and buckles for signs of worn out parts - Be aware of any MAX mark indications before adjustment 	Weekly	<ul style="list-style-type: none"> - Check that all castors are moving freely and that all wheel locks/brakes are easily activated
Frequency	Description								
1 st Day	<ul style="list-style-type: none"> - Read the instructions for use (IFU) carefully before use and store them in a safe place for future reference - Store the enclosed tools on/at the product 								
Daily	<ul style="list-style-type: none"> - Use a dry cloth to clean the product - Visual inspection. Repair or replace any dam-aged or worn-out parts - Ensure that all fasteners are in place and fit-ted correctly - Check all fittings and buckles for signs of worn out parts - Be aware of any MAX mark indications before adjustment 								
Weekly	<ul style="list-style-type: none"> - Check that all castors are moving freely and that all wheel locks/brakes are easily activated 								

Crocodile	
	<ul style="list-style-type: none"> - Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use - Disinfect the product. please check the disinfection section in this User Guide for more information
Monthly	<ul style="list-style-type: none"> - All nuts and bolt on this product must be checked and tightened to avoid unnecessary failures - Oil the swing-away parts. We recommend you use a professional lubricant system
Annually	<ul style="list-style-type: none"> - Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty
Cleaning description	<p>This product can be washed at 60°C in 10 minutes using a mild detergent in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Clean the accessories separately.</p> <p>Before washing Remove the accessories and clean them separately:</p>
Disinfection description	<p>The product without accessories can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.</p>
Dimensions	<p>https://www.etac.com - Select product and go to the variants section.</p>
Mass	<p>https://www.etac.com - Select product and go to the variants section.</p>
Used in combination with other products	<p>N/A</p>
Warning about dangerous combinations	<p>The CE mark must be removed when:</p> <ul style="list-style-type: none"> • Rebuilding the product • Using the product outside the intended purpose and declaration of conformity • Using other than original R82 spare parts and fittings <p>When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.</p> <p>Applicable requirements examples: Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety</p>
List of accessories, detachable parts and materials	<p>https://www.etac.com - Select product and go to accessory section</p>
Programmable controller information	<p>N/A</p>
Operator control adjustments	<p>N/A</p>
Folding and dismantling	<p>Foldable with the possibility of separating frame and handlebar(s)</p>
Transport instructions	<p>Unoccupied device is suitable for land and air transport</p>
Sound power level	<p>N/A</p>