

**Thank you for choosing an Etac product.**

If in doubt, please contact Etac Customer Service or local distributor for more information and guidance.  
For more information about training and the use of these products, contact the Etac transfer experts - [www.etac.com](http://www.etac.com).

To avoid accidents and injury when moving and handling the patients, read the manual carefully.

If the customer makes adjustments, repairs or combinations not predetermined by Etac, the Etac CE certification and Etac warranty will no longer be valid.  
If in doubt, please contact Etac.

In case of an adverse event occurred in relation to the device, incidents should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

**Expected lifetime:** the device has an expected service life of 1 to 2 years under normal use. The service life of the product varies depending on usage, frequency, loads and how often and how it is washed and tumble dried.

**Warranty:** Two-year warranty on material and manufacturing defects, provided that the product is used correctly.



## EXPLANATION OF SYMBOLS FOR WASHING INSTRUCTIONS



Do not wash



Do not tumble dry



Wash/temperature celsius



Ironing, low temperature



Wash/temperature celsius



Do not iron



Do not bleach



Do not dry clean



Tumble drying  
low temperature, max 55°C



This product can be cleaned with a solventfree  
cleaning agent with a pH level between 5-9, or  
a maximum 70% disinfectant solution.  
+ Wiping

## First inspection:

- Is the packaging intact?
- Read the label on the packaging and check the article no. and product description
- Check that Short Instruction is enclosed – updated User Manual/Instruction for use can be downloaded from [www.etac.com](http://www.etac.com) or contact your Etac Customer Service or the local dealer.
- Check the label on the product - does it include article no., product description, lot/batch no., cleaning instructions and supplier name?

## Periodic Inspection:

- Make sure that your Manual transfer system always is in a perfect condition.
- After wash – always check materials, stitching, handles, buckles.
- If the product shows signs of wear and tear, it must be removed from service immediately.



### Visual inspection/Check the product:

Ensure that the materials, seams, stitching, handles, buckles are intact/faultless.



### Stability test:

Try to bend the product verify that it feels firm, stable and solid.



### Mechanical load or stress/Test handles:

Draw hard in the handles in opposite directions and control the material, seams and stitching.

Test buckles:

Lock the buckle and draw in opposite directions, control the buckle and the stitching.



### Test low/high friction:

Place the product on a firm surface or a bed, place your hands on the material and test the friction by pushing your hands down into the product.

Low friction – slides effortlessly

High friction – no sliding/moving

## Check:

Manual transfer aids with handles, straps and buckles:



Manual transfer aids with for sitting and lying transfer and with high or low friction:



Production year, month and day



The product can be scrapped in accordance with national regulations



Read the user manual