
User manual

Molift Quattro

molift[®]
by Etac

BM22099 - 2024-06-10 - Rev. 1.0

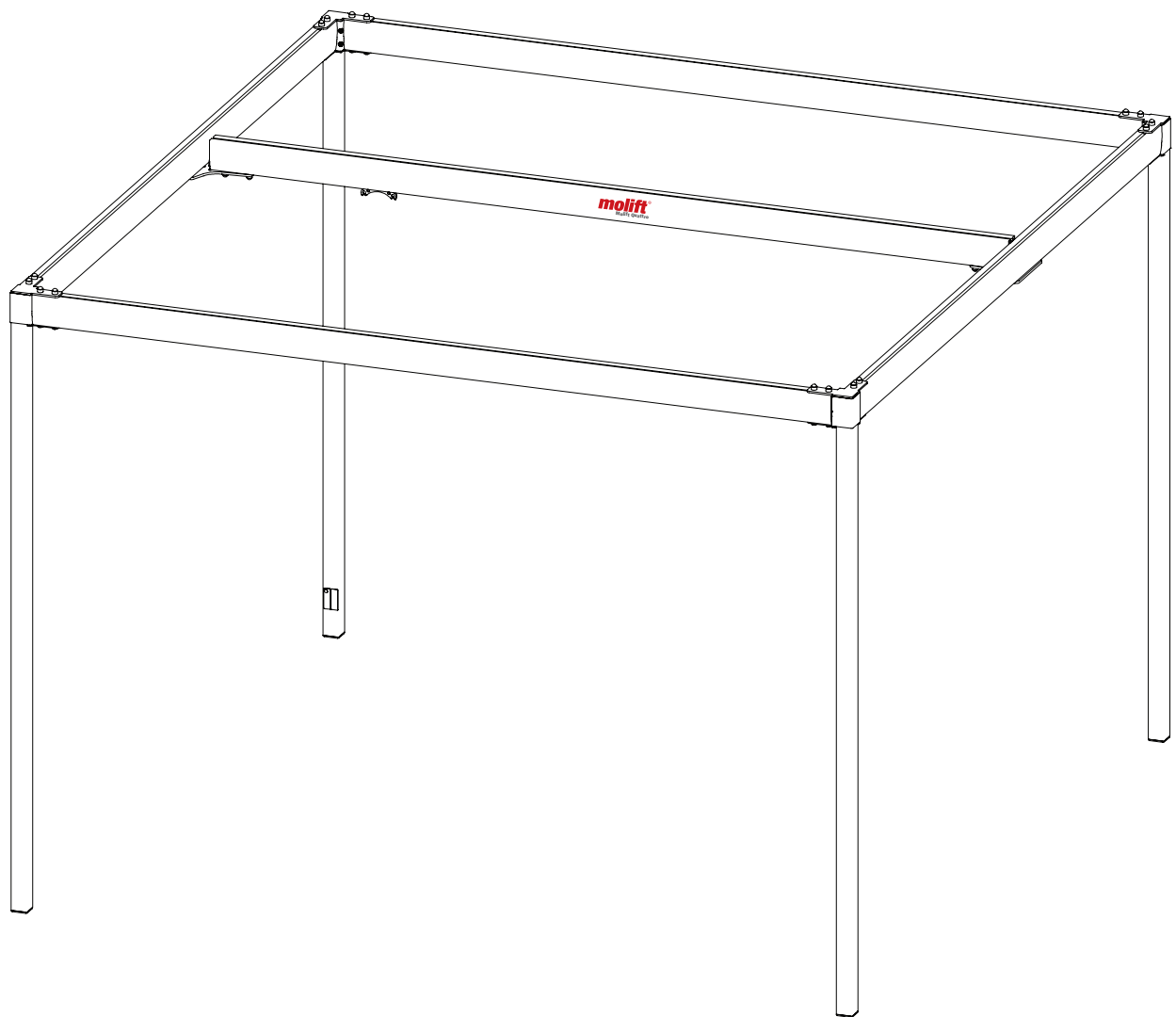www.etac.com

MD

Index

User manual - English	3
Bruksanvisning - Svenska	15
Brukermanual - Norsk	27
Brugsvejledning - Dansk	39
Käyttöohje - Suomi	51
Gebrauchsanweisung - Deutsch	63
Handleiding - Nederlands	75
Manual d`utilisation - Francais	87
Manuale d'uso - Italiano	99
Manual de usuario - Español	111

English Manual

EN

Content

Molift Quattro.....	4
Molift Quattro components	4
About Molift Quattro	4
General.....	5
Declaration of conformity.....	5
Conditions for Use	5
Warranty	5
Product identification.....	5
Technical Data.....	6
Assembly	7
Supplied parts	7
Mounting of Traverse boom.....	7
Mounting Quattro Frame	7
Tip: Perpendicular frame	8
Mounting of legs, Ceiling height > 2,4 m	8
Mounting of legs, Ceiling height = 2,4 m	9
Checklist before use.....	10
How to use Molift Quattro	10
Lifting user	10
Transport and storage.....	10
Accessories	11
Adjustable foot.....	11
Battery charger.....	12
Molift Air.....	12
Molift Nomad.....	12
Boom	12
In Rail Charging System.....	12
Maintenance	13
Cleaning and disinfection.....	13
Reconditioning.....	13
Recycling	13
Spare parts.....	13
Periodic Inspection	13
Periodic Inspection Scope.....	13
Troubleshooting	14

Important

This User Manual contains important safety instructions and information regarding the use of the Quattro and accessories.

In this manual, the user is the person being lifted. The assistant is the person operating the hoist.

**Warning!**


This symbol indicates important information related to safety. Follow these instructions carefully.

**Read User Manual before use!**

It is important to fully understand the content of the user manual before attempting to use the equipment.

Visit www.etac.com for download of documentation to ensure you have the latest version.

Molift Quattro components:

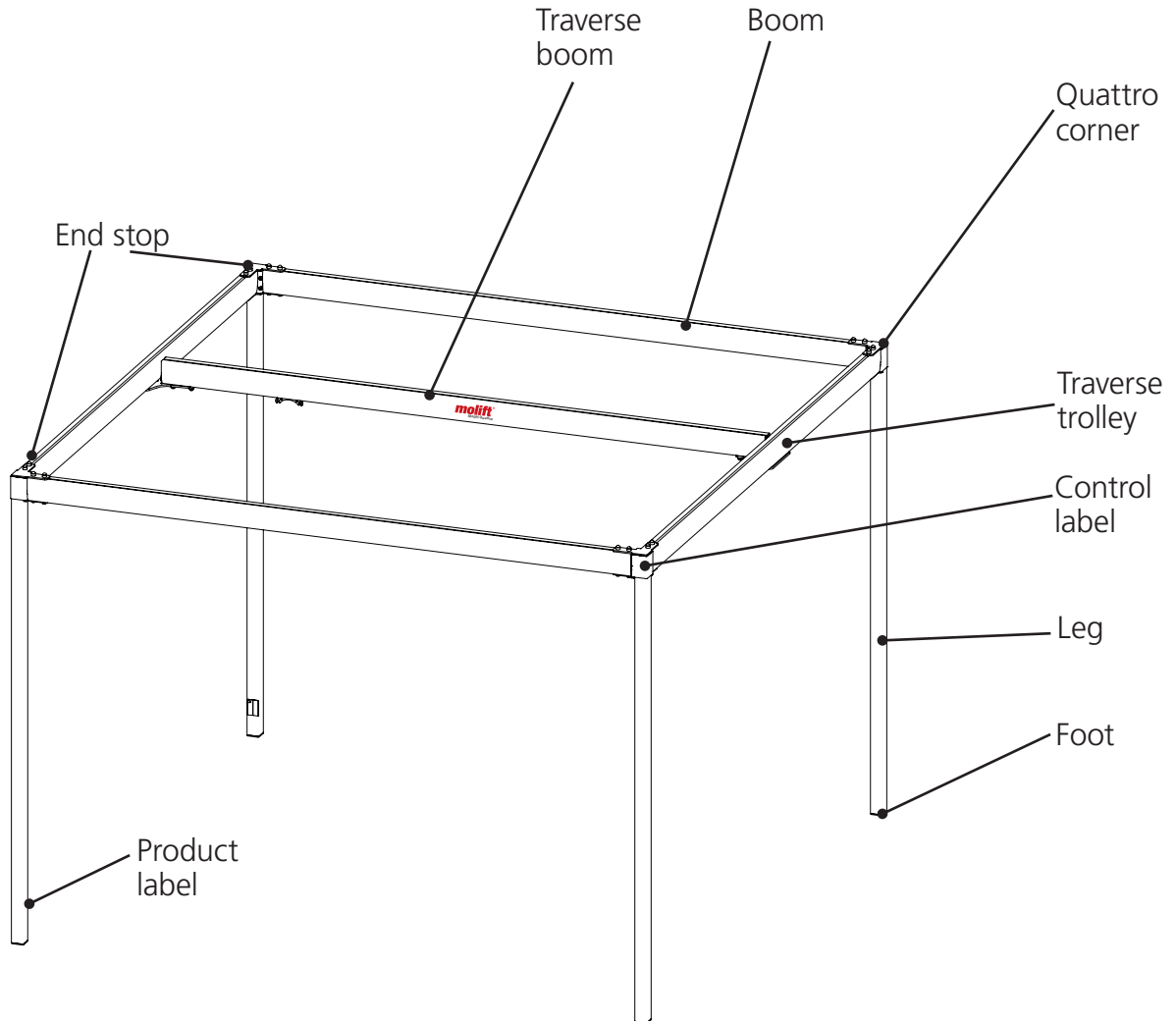About Molift Quattro

Molift Quattro is a gantry intended for use with Molift Air and Molift Nomad hoists. The purpose is for lifting and transferring a person from/to bed, floor, chair, wheelchair or toilet with a sling. The Molift Quattro is made of light-weight materials. Traverse ranges from 2 to 3,5 meters and the lifting capacity is 300 kg (661 lbs). It is ideal for

hospitals, and institutions, where permanent installations are not possible or desirable, but can also be used in private homes. The Molift Quattro should only be used indoors on flat surfaces. Such as bedroom, bathroom, living room etc.

General

EN

Declaration of conformity

The Molift Quattro and related accessories described in this user manual are CE marked in accordance with EU Council Directive 93/42/EEC concerning medical devices, class 1, complies to standard EN ISO 10535:2021.

Conditions for Use

Hoisting and transferring a person may contain an element of risk. A risk assessment should be carried out prior to use and only trained personnel should use the equipment as described in the user manual.

Modifications and use of components made by other manufacturers.

We recommend only using Molift components and spare parts. Declaration of conformity is not valid and Etac is not responsible for warranty if any modifications are made to the product. Etac shall not be liable for faults or accidents that can occur when using components made by other manufacturers.

Only certified personnel are allowed to perform repairs.

Only suitable trained people should install this equipment.
The work contains handling of heavy loads.


Warranty

2-year warranty against defects in workmanship and materials of our products.
For Terms and conditions, see www.etac.com.

Product identification

Product label

The Product label contain article number and production date.

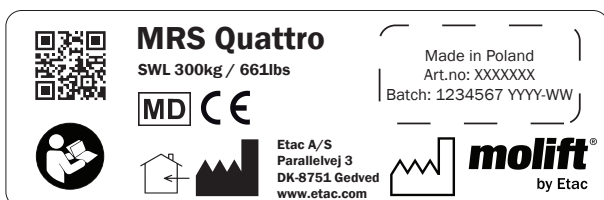molift
Molift Quattro

Warning labels and Symbols

Symbols used on the product, explained in more detail:

CE marked

Refer to user manual

Indoor Use only

Manufacturer

Date of manufacture

Medical Device

Safe Working Load (SWL)

300 kg (661 lbs)

Weight of unit

Boom Kg/m: 4,2 kg/m/8,8 lbs/m
 Leg with adjustable foot: 8,5 kg/18,7 lbs
 Corner: 2,7kg/6,0 lbs
 Total weight (Complete unit):
 110,95kg (3x3m)
 131,55kg (4x4m)

Available boom lengths (A):

3m or 4m

Dimensions:




Width (D): 3,16 - 4,16m
 Height (B): 2,33m (2,33-2,38m)
 with adjustable foot
 Depth (E): 3,16 - 4,16 m
 Max internal space (AxC):
 3x3m or 4x4m

Material:

Steel, Anodized Aluminum, Plastic

Expected Lifetime:




The Molift Quattro has an expected lifetime of 10 years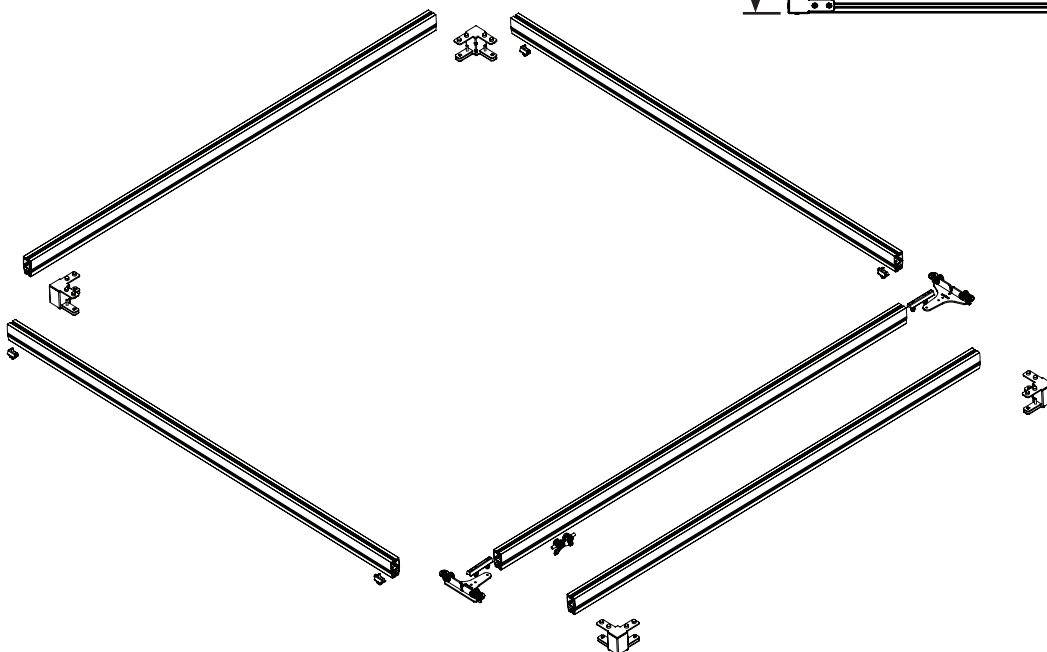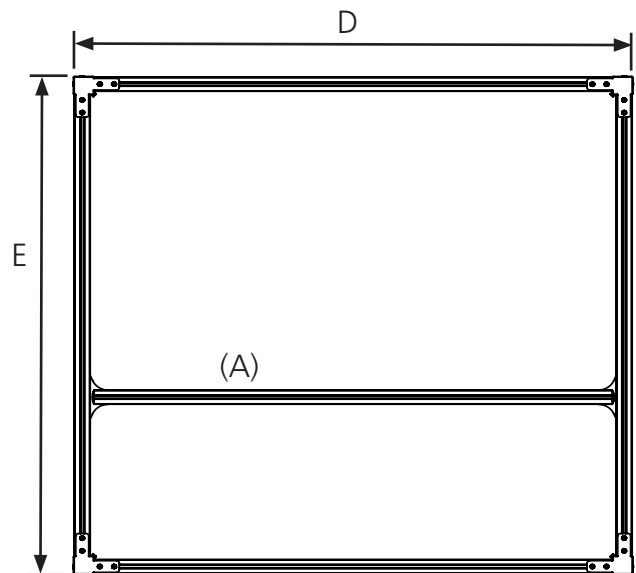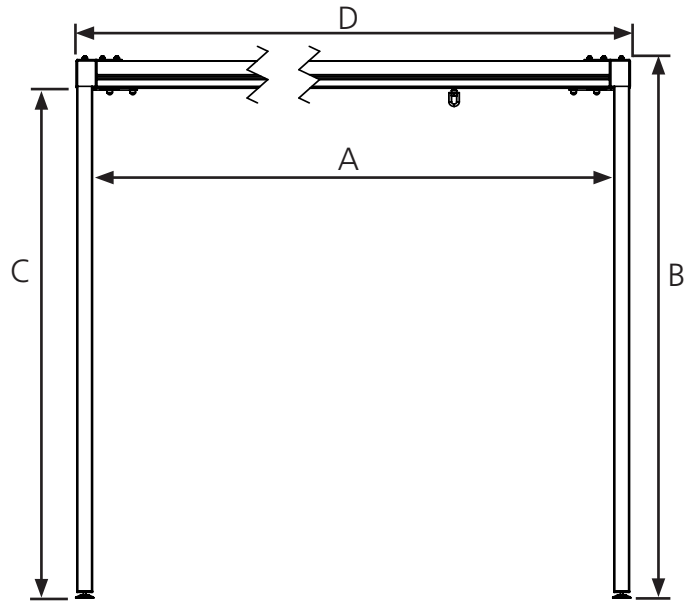

Assembly EN

Supplied parts


Make sure the following parts are included in the package.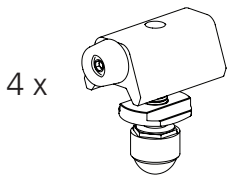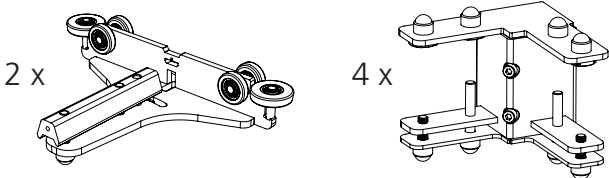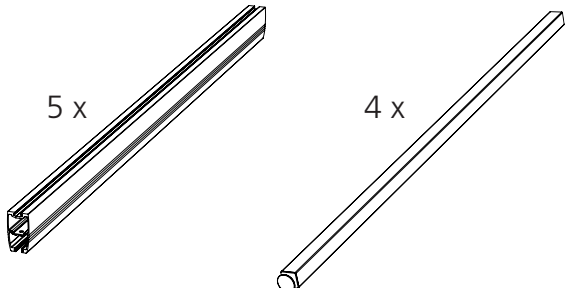


Beware of sharp edges at the ends of the boom.

Mounting of Traverse boom

1. Lay boom on the floor.
2. Insert trolley for the hoist into the boom.
Trolley (not included) and must be ordered separately.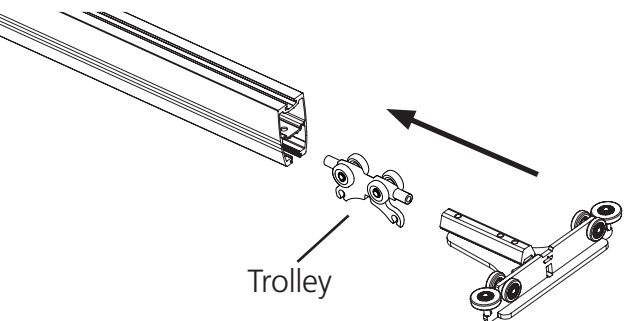


3. Mount the traverse trolleys in each end of the traverse boom.
4. Tighten the bolts loosely.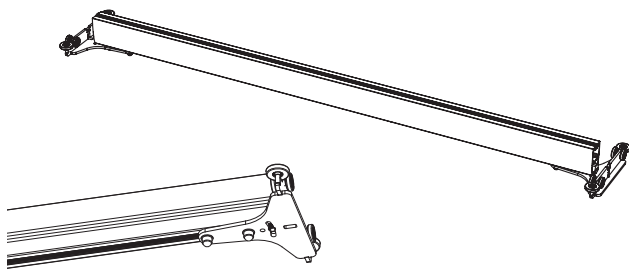


Mounting Quattro Frame

Tip: To facilitate the installation, make 4 support blocks, approximately 150 mm high.


1. Make sure you have free space of aprox. 3x4 meters in the room where the Molift Quattro will be mounted.
2. Place the support blocks on the floor.
3. Place two Molift Booms on support blocks as picture below shows.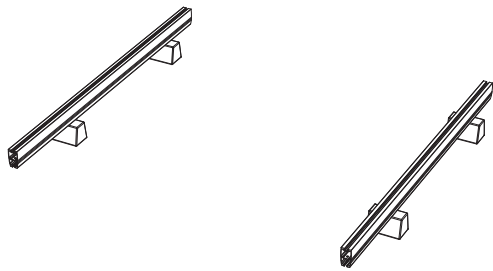


4. Slide in the Traverse Boom into a mid position. Slide the end stop (4 pcs) into the booms. Fasten end stops temporary during assembly. Must be fastened properly with 8mm Allan key after adjustment.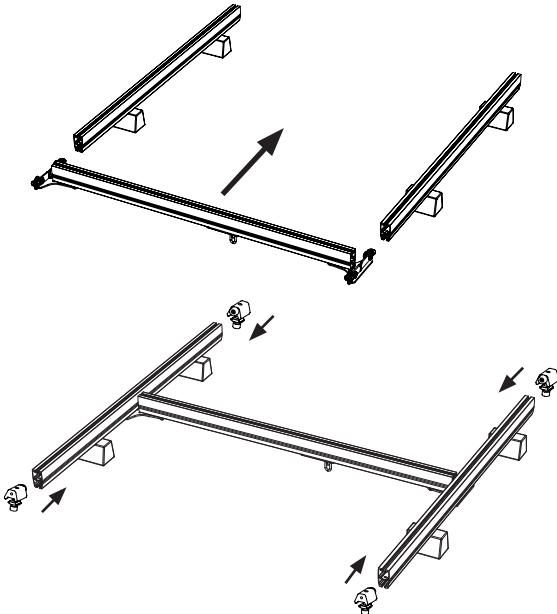



5. Mount two Quattro Corners to each of the other booms.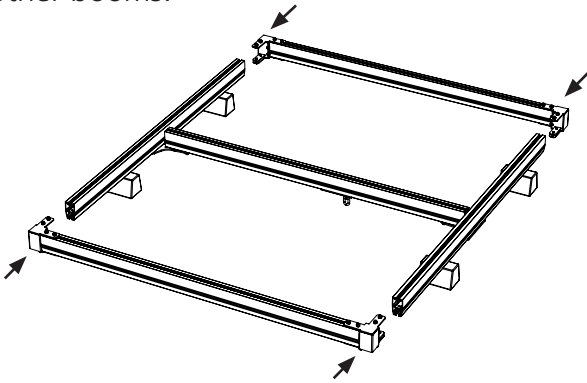

6. Attach booms with mounted corners on to the other two boom.
7. Tighten all the M10 bolts and nuts with a spanner. M10 bolts torque 35 Nm.

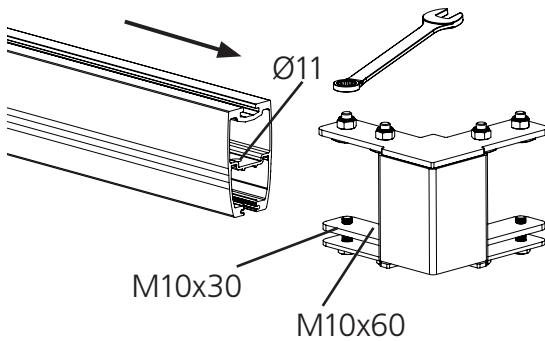
8. Make sure M10x60 bolts is fixed in the Ø11 hole in all 4 pcs booms.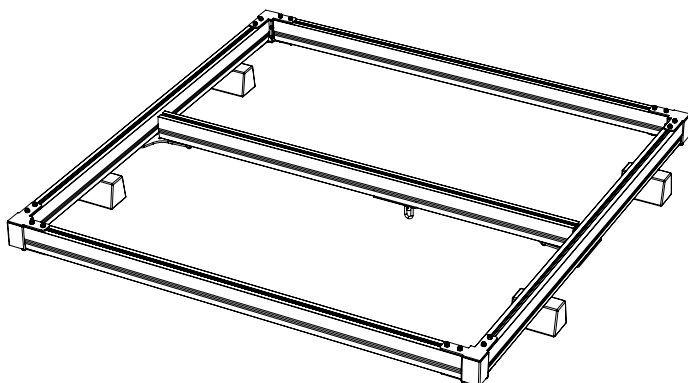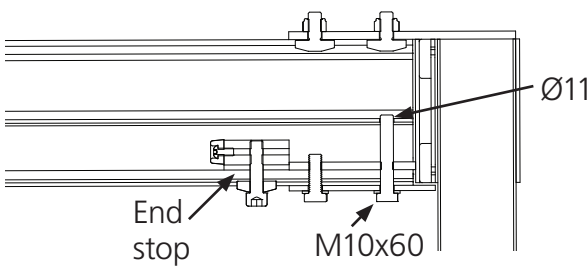

Tip: Perpendicular frame.

If possible, use a tape measure to check that the Molift Quattro Frame is perpendicular.

1. Measure the diagonal length corner-to-corner. The length shall be the same both ways.
2. If not; loosen the bolts and adjust.
3. When the readings is the same the frame is square (or rectangle).
4. Tighten all bolts and nuts.

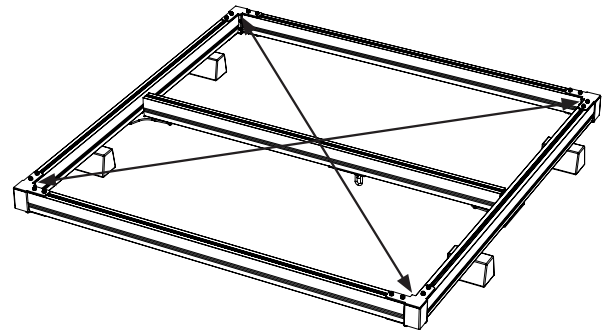
Secure that bolts are tightened properly. Loose bolts may cause collapse of the Molift Quattro when it is raised!

Warning for lifting of heavy loads. Dropping the frame can cause injury.

Mounting of legs, Ceiling height > 2,4 m

1. Lift the Molift Quattro frame at one side and fit legs in the corners.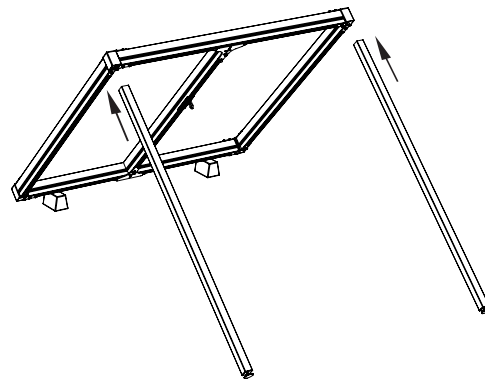


Note! Secure the boom and trolley from running free. Use a strap or tape.

Assembly

EN



2. Tighten the two bolts with Allan key 5mm. Use torque 15 Nm.

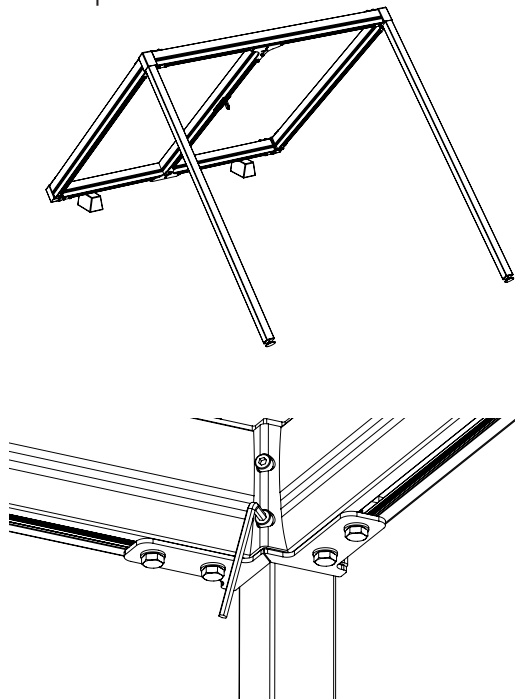3. Lift the frame to an upright position and attach legs at the opposite corner



Before raising Molift Quattro, secure that you use rigid supports preventing Quattro from falling during the work.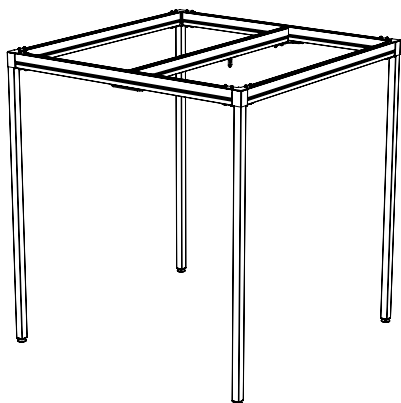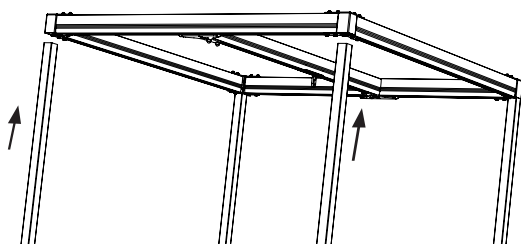


Mounting of legs, Ceiling height = 2,4 m

Start from step: Perpendicular frame.


1. Detach one boom and keep the corners on.
2. Move the traverse boom to a position close the open ends.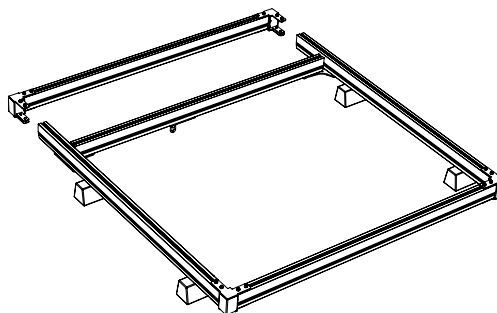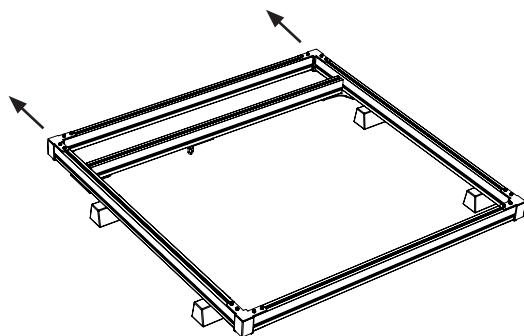


3. Secure the traverse boom and trolley from moving before raising the frame.
4. Attach legs to boom with corners.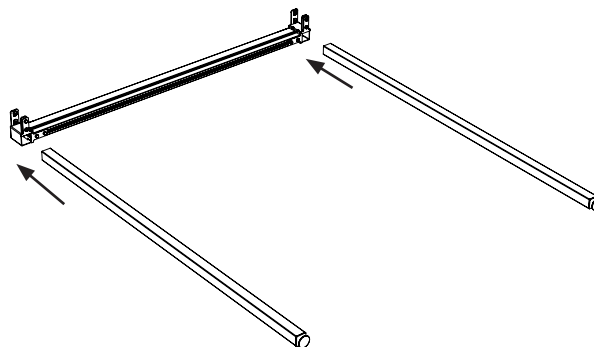

5. Raise up the open frame.
6. Attach legs to corners.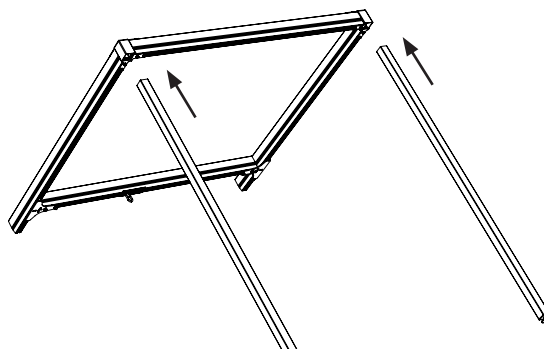


EN **Assembly**

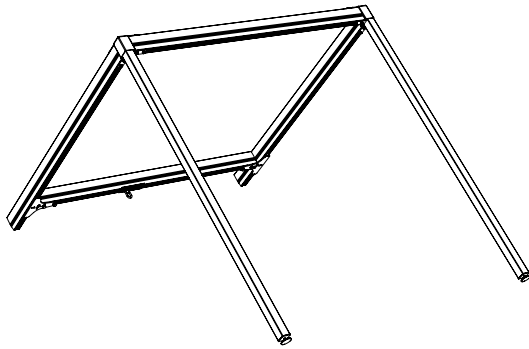
1. Tighten the bolts with an Allen key.

 Before raising Molift Quattro, secure that you use rigid supports preventing Quattro from falling during the work and that the boom is secured.

- 2. Rise up the frame with legs to an upright position.
- 3. Attach the boom with legs to the open frame.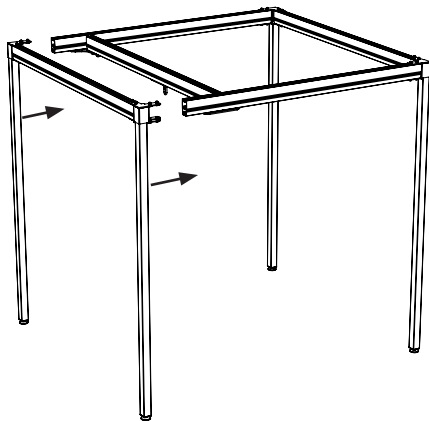

- 4. Tighten the bolts and nuts.
- 5. Insert the plastic foot.
- 6. Release the traverse boom.

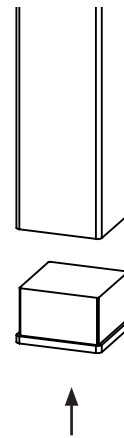Checklist before use

 **Don't start using Molift Quattro before completing control according to checklist after assembly.**

Use this checklist to verify that the Molift Quattro is properly installed and can operate correctly and safely before use.

- 1. Make sure all bolts in all corners are properly fastened.
- 2. Make sure all four bolts on boom are properly fastened.
- 3. Make sure the Quattro is levelled correctly for easy movement of boom and hoist trolley.

EN **How to use Molift Quattro**

Lifting user

Read the user manual for Molfit Air or Nomad before lifting patient.

 **Never lift the user higher than necessary to carry out a lift.**

 **Never leave a user unattended in a lifting situation.**

Transport and storage


Molift Quattro can be stored and transported under temperatures between -25 - 70 °C and Relative Humidity between 15 - 93 %.

Accessories


EN

Adjustable foot

1. Remove the feet from the Molift Quattro.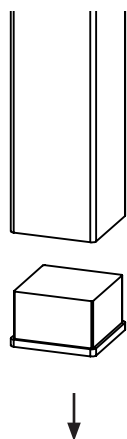

2. Attach the adjustable foot.
Art no: 1610200

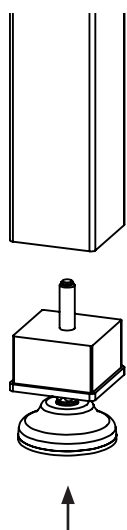
3. Adjust the levelling feet for horizontal position of Molift Quattro.

Unscrewing the foot too much can cause the foot to come loose.

Battery charger


Molift Nomad and Molift Air 200/350 is charged through the handcontrol. The handcontrol charger can be mounted on the Molift Quattro.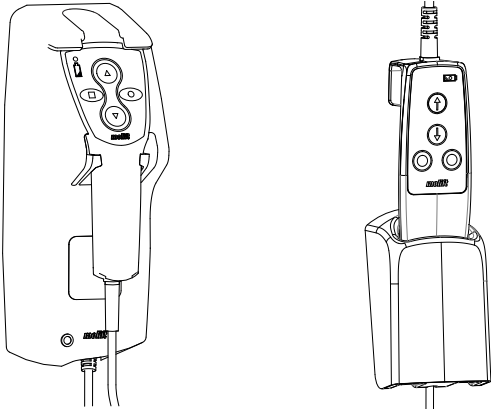

Charger Molift Nomad.
Charger Molift Air 200/350.

Molift Air

Molift Air 200/350 is suitable for use with Molift Quattro.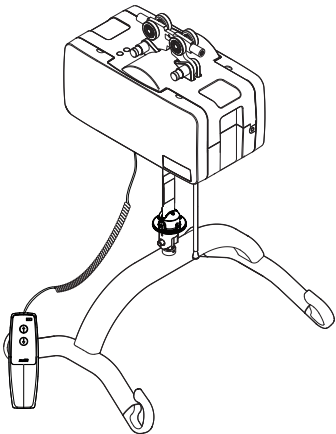


Molift Air is available with or without propulsion, with 205 or 300 kg SWL.

- Molift Air 200 or 350
- Molift Air 200 or 350 Propulsion
- Molift Air 200 IRC
- Molift Air 350 IRC

Molift Nomad

Molift Nomad portable hoist is suitable for use with Molift Quattro.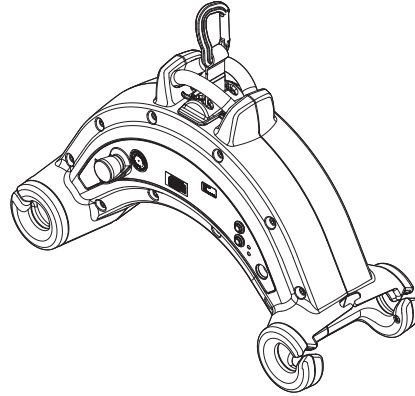


Molift Nomad is available with 205 or 255 kg SWL.

- Molift Nomad 205 kg SWL
- Molift Nomad 255 kg SWL

Boom

Molift Quattro can be delivered with 2 different boom lengths from 3 - 4 meters.

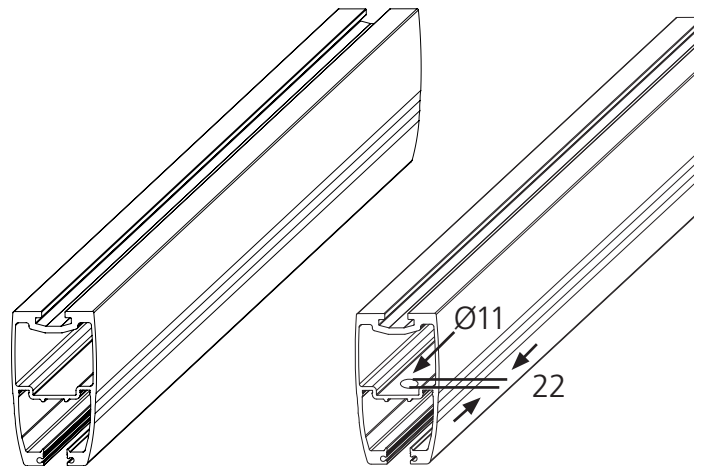Boom for Molift Quattro:

Art. no.: 1610030 - 3 m

Art. no.: 1610094 - 4 m

In Rail Charging System

Possible to use the in rail charging system with the Molift Quattro. Contact supplier or www.etac.com for more info.

Maintenance

EN

Cleaning and disinfection

Clean on a regular basis. Clean surfaces with a damp cloth using an appropriate pH-neutral detergent. Do not use solvents or strong liquids, this may damage surfaces. Disinfection when needed; use isopropyl alcohol. Avoid abrasive cleaning products.

Make sure not to damage or remove labels when cleaning.

Reconditioning

Molift Quattro can be re-used. Follow assembly instruction to dismount and re-assembly.

Recycling

Molift Quattro can be disposed of as metal waste without battery charger. Battery charger must be removed and disposed of as Electrical equipment.

Spare Parts

A list of spare parts is available on request.

Periodic Inspection

Periodic Inspection Scope

Periodic inspection is a visual examination (particularly of the Molift Quattro structure, bolts and person-support devices according to Periodic Inspection Report for Molift Quattro. This can be found on www.etac.com.


In the event of danger to safety, the Molift Quattro shall immediately be taken out of service and marked clearly with "out of order" and shall not be used until the Molift Quattro is repaired

Periodic Inspection shall be performed at least once a year or more frequently if required by local requirements. The inspection should be performed by service personnel authorized by Etac. Contact Etac for training and authorization or recommendation of an approved service partner.

When performing a periodic inspection, the inspector shall fill out the inspection report for Molift Quattro. The reports should be retained by the person(s) responsible for servicing the hoist. If the inspection reveals defects and damages, the owner shall be notified and a copy of the report should be sent to Etac, patienthandling@etac.com.

When periodic inspection is completed the inspector shall mark the Molift Quattro with a sticker on the control label showing the date when periodic inspection is performed and a label (with 6-digit ID from Etac or distributor) showing who has performed the periodic control. The control label can be found on the post, and this will then indicate when next service should be performed.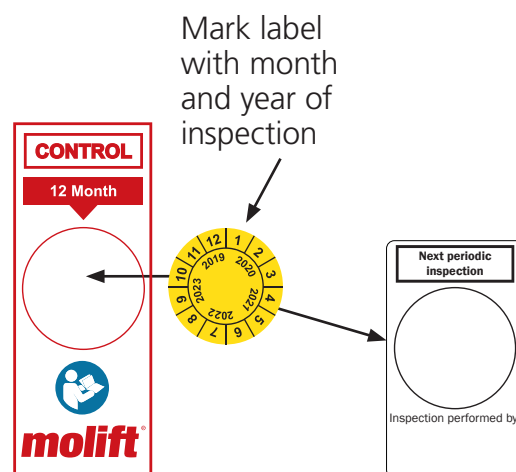

EN

Troubleshooting

Symptom	Possible Cause/Action
Wobbly between leg and frame.	Check bolts in corner. Make sure bolts is properly tightened.
	Adjustable foot adjusted up too much.
Wobbly between Traverse boom and frame.	Check bolts between bracket and boom. Make sure bolts are properly tightened.
	Control bolts between bracket and post. Make sure bolts is properly tightened.
Molift Quattro jams.	Loose bolts on traverse boom.
	Damaged wheels on traverse boom.
	The frame is not orthogonal.

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Etac Ltd.
Etac A/S